


Flo-Rester®

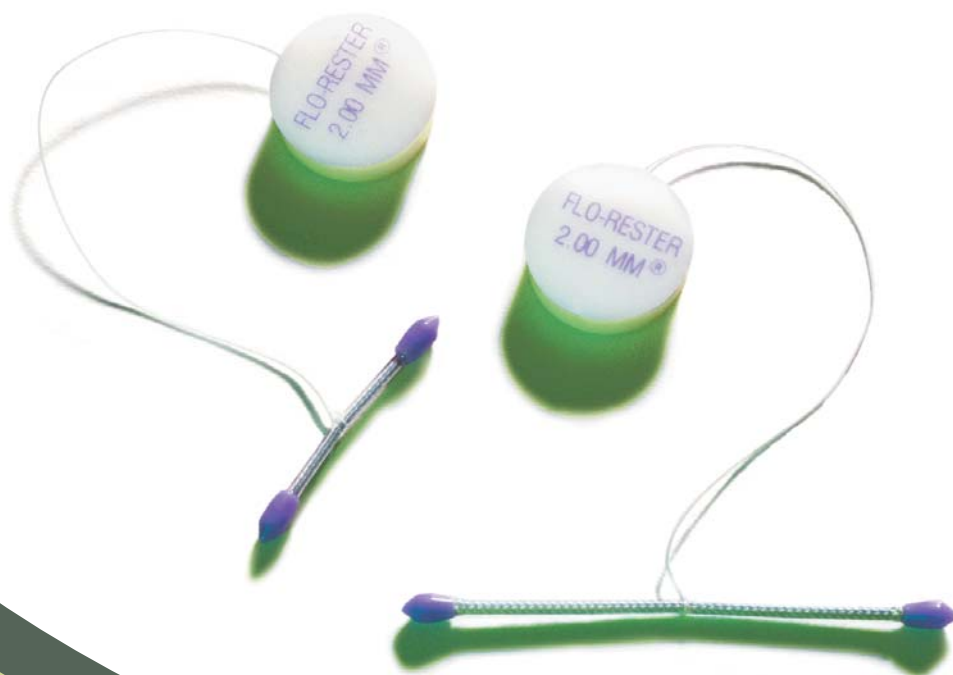

Designed to be a  
safe, simple  
atraumatic way to  
control bleeding  
during bypass  
surgery and  
vessel repair.

## **Flo-Rester®**


### Internal Vessel Occluder

**Synovis®**  
Surgical Innovations


# Flo-Rester®

For more than 20 years, surgeons have relied on the Flo-Rester® Internal Vessel Occluder to provide a safe, simple, atraumatic method of controlling bleeding and stenting vessels during bypass grafting and vessel repair.


The Flo-Rester is a gentle, easy-to-use internal vessel occluder composed of two radiopaque silicone bulbs secured on the distal ends of a very flexible shaft. A tab, also radiopaque, is attached by a suture line tether to the mid-point of the shaft forming a “T”-shaped configuration.

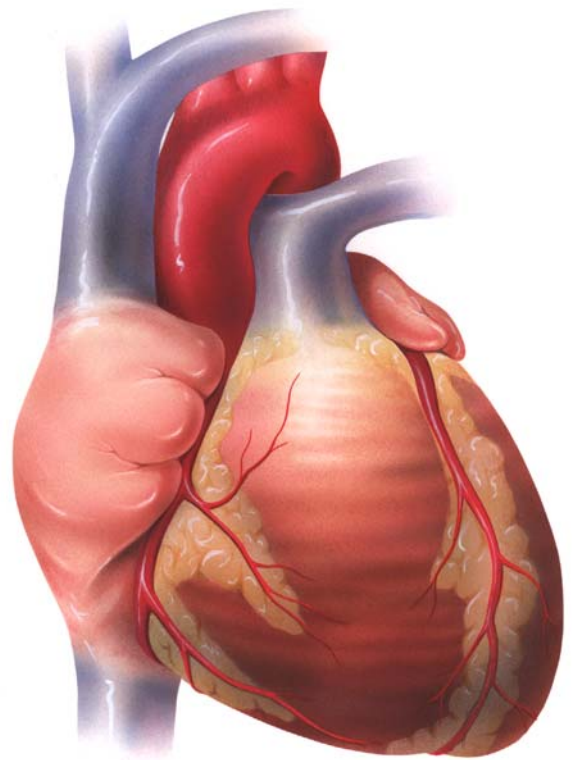


## Using Flo-Rester

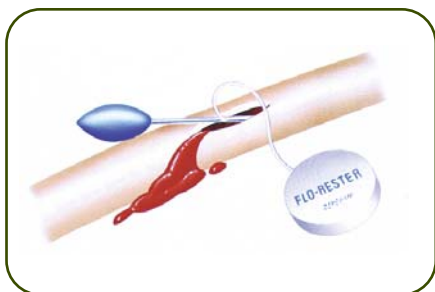

Easy Insertion

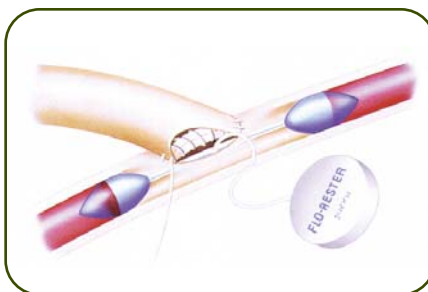

Vessel Occlusion  
and Stenting

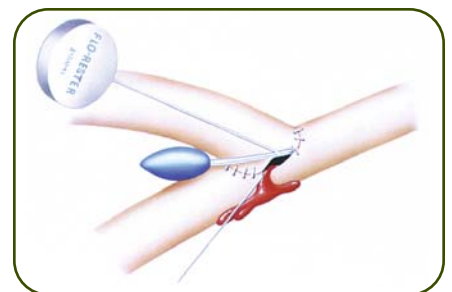

Simple Retraction  
and Easy Removal

## **Surgical Applications**

Coronary bypass surgery and peripheral vascular bypass surgery:

- Femoral-popliteal bypass
- Tibial bypass
- Peroneal bypass
- Dorsal-pedal bypass


## **Clinical Benefits**

- Provides bloodless field for excellent visualization at the site of anastomosis
- Stents the vessel for improved presentation, better needle placement and enhanced suturing
- Eliminates external pressure on the vessel (e.g., clamps and vessel loops) preventing intimal damage to the vessel
- Flexible shaft and atraumatic bulbs allow for easy insertion and removal


## **A Convenient Choice**

- Wide range of bulb diameters<sup>†</sup> (1.00 mm to 4.50 mm) allows a proper fit based on the internal diameter of the vessel
- Supplied sterile and ready to use in 12 mm or 30 mm shaft lengths

## Ordering Information<sup>†</sup>

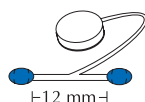

12 mm Sizes

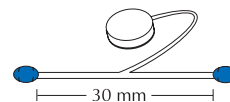

30 mm Sizes

| PRODUCT NUMBER | BULB DIAMETER | UNITS PER BOX | PRODUCT NUMBER | BULB DIAMETER | UNITS PER BOX |
|----------------|---------------|---------------|----------------|---------------|---------------|
| 50100          | 1.00 mm       | 5             | 60100          | 1.00 mm       | 5             |
| 50125          | 1.25 mm       | 5             | 60125          | 1.25 mm       | 5             |
| 50150          | 1.50 mm       | 5             | 60150          | 1.50 mm       | 5             |
| 50175          | 1.75 mm       | 5             | 60175          | 1.75 mm       | 5             |
| 50200          | 2.00 mm       | 5             | 60200          | 2.00 mm       | 5             |
| 50225          | 2.25 mm       | 5             | 60225          | 2.25 mm       | 5             |
| 50250          | 2.50 mm       | 5             | 60250          | 2.50 mm       | 5             |
| 50275          | 2.75 mm       | 5             | 60275          | 2.75 mm       | 5             |
| 50300          | 3.00 mm       | 5             | 60300          | 3.00 mm       | 5             |
| 50350          | 3.50 mm       | 5             | 60350          | 3.50 mm       | 5             |
| 50400          | 4.00 mm       | 5             | 60400          | 4.00 mm       | 5             |
| 50450          | 4.50 mm       | 5             | 60450          | 4.50 mm       | 5             |
| STARTER PACKS* |               |               | STARTER PACKS* |               |               |
| 50001          | Small Sizes   | 6             | 60001          | Small Sizes   | 6             |
| 50002          | Large Sizes   | 6             | 60002          | Large Sizes   | 6             |

\*Starter Packs contain one each of the six smallest (1.00 mm - 2.25 mm) or six largest (2.50 mm - 4.50 mm) sizes.

<sup>†</sup> All sizes nominal

Flo-Rester is for single-use only and cannot be resterilized. Flo-Rester occluders can also be made in various lengths and bulb diameters to accommodate surgeon preference. These special orders require additional charges and manufacturing time.

Contact your sales representative or Synovis Surgical Innovations for ordering information, in-service support materials, and technical specifications.

# Synovis®

Surgical Innovations

A Division of Synovis Life Technologies, Inc.

2575 University Ave. W.  
St. Paul, MN 55114-1024 USA  
651.796.7300 800.255.4018  
651.642.9018 (fax)  
synovissurgical.com

Distributed by:

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician. For complete instructions for use, storage, warnings, indications, contraindications, precautions, adverse reactions, and disclaimer of warranties, please refer to the insert accompanying each Flo-Rester product.

Flo-Rester is a registered trademark of Synovis Life Technologies, Inc.

94007D 03/05 © 1997 Synovis Surgical Innovations. All rights reserved. Printed in the U.S.A.

CE  
0086