

Simple. Effective.

mediCipio A

mediCipio A


The patented, two-layer structure of mediCipio A combines the excellent haemostatic effectiveness, absorbency and adhesion of oxidized regenerated cellulose (ORC) with the stability and biocompatibility of porcine collagen, without using human fibrinogen and thrombin.

Simple.

In use.

Effective.


In hemostasis.

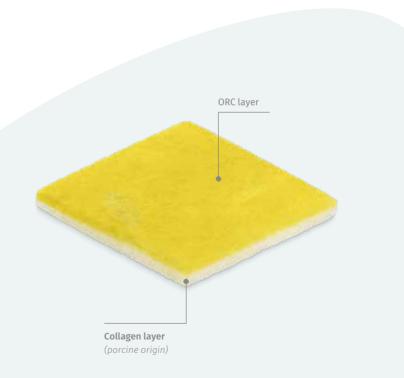


2

Easy to use thanks to the best material

- Soft and flexible modelling of the wound surface immediately after removal (Fig. 1)
- No additional moistening needed
- Adheres to the wound, not to gloves or instruments
- With mediCipio A, all risks associated with the use of human blood components are eliminated a priori

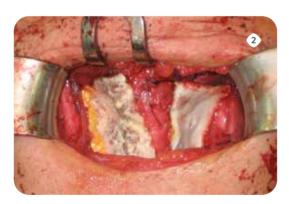

Excellent visibility in the surgical field

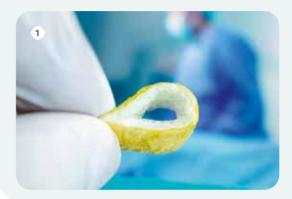

Freely formable in dry state

- Upon contact with tissue fluid, the fleece does not discolour, but becomes semitransparent and allows the evaluation of haemostasis (Fig. 2)
- Available in different sizes and easily cut to size sterilely if required
- Suitable for use in a trocar



Doctors put in so much effort to per-


form at their best – the haemostatic agent should not let them down.

Ensuring medical success effectively

- Safe and reliable haemostasis
- Promotion of wound healing and ideal biocompatibility through porcine collagen
- No increase in volume can therefore be used even close to nerves
- No detachment of fragments from the fleece
- No allergenic additives, no risks from human thrombin or fibrinogen
- mediCipio A is completely resorbed within a few weeks

6

Efficacy study Effective

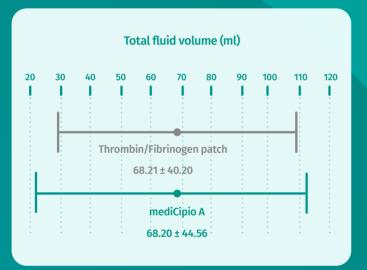

Primary endpoint: Total fluid volume at the time of removal of the Redon drainage system following thyroidectomy

Evidently effective: Best haemostatic quality

In a prospective and randomized clinical comparison study with 150 patients it was confirmed that mediCipio A is as effective in inducing haemostasis as a product based on human thrombin and fibrinogen. Safe haemostasis was also confirmed when using platelet aggregation inhibitors.

Simply advantageous as a medical device

- From raw material to end product Made in Germany
- Available in various sizes
- Can be stored at room temperature
- Cost savings through process optimization
- No mandatory documentation according to transfusion law
- Attractive pricing

Size	Content	Pharmaceutical central number/ code
1 x 5 cm	5 items	1417 4850
2.5 x 3 cm	5 items	1417 4867
5 x 5 cm	2 items	1417 4873
5 x 10 cm	2 items	1417 4904

Each individually sterile-packed. Can be cut sterile if required.

Further information and orders: Tel.: +49 371 90 96 83 8



mediCipio

is a brand of

medichema GmbH

Weststraße 57 09112 Chemnitz Germany

Tel.: +49 371 90 96 83 8 Fax: +49 371 90 96 83 9

info@medichema.com
www.medichema.com